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
ROTORES CON PERFILES ESPECIFICOS

VETERINARY ROTOR PROFILE & ORDER CODE

Analyte (Full Name)	ALB	ALP	ALT	AMY	AST	CO	CHE	Cl	CK	Cre	GLU	GGT	K+	LDH	LPX	Mg+	Na+	PHOS	TBA	TB	TC	TCCO	TG	TP	UA	UN	UCRP	PI	APPI	IT	HS	SMA	AVP	VC	OLG	
Comprehensive Test #AW00704 13	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Health Check #AW00085 13	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Liver Function #AW00067 9	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Kidney Function #AW00198 8	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Electrolyte #AW00234 7	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Primary Diagnostic #AW00686 10	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Emergency Diagnostic #AW00254 10	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Pre-operation #AW00235 9	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Canine Inflammation #AW00470 5	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Coagulation #AW00650 4	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Feline Inflammation #AW00738 6	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*

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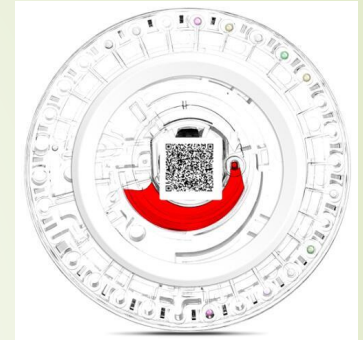whole blood
plasma
serum

FELINE INFLAMMATION

- No need external centrifugation, only 3 steps
- Only 100ul, all sample types, no carryover

FELINE INFLAMMATION

f-SAA(Feline serum amyloid A) is an acute phase protein, secreting in the acute phase of inflammation, increasing in the onset and recurrence. Especially in the early stages of inflammation and tissue damage, the concentration of SAA will increase more, significantly increasing in cases including keratoconjunctivitis, periodontitis, stomatitis, trauma, acute laryngitis, acute pancreatitis, peritonitis, empyema, ulcers/dermatitis and cystitis, meanwhile SAA is an effective marker for assessing cats with pancreatitis as well. Therefore, SAA is an important inflammatory indicator and prognostic marker for cats.

- (f-SAA) Feline serum amyloid A
- (ALT) Alanine aminotransferase
- (TBA) Bile Acids
- (BUN) Urea nitrogen
- (TP) Total protein
- (CK) Creatine Kinase

• Freeze-drying reagents technology, with one year shelf life
• Full-auto operation, no need to manually inject diluent
• Advanced microfluidic technology, avoid carryover
• Real-time QC system, ensure accurate results

*No requirement for professional veterinarian to operate

1 Inject 90-120ul sample 2 Insert reagent disc 3 Read report after 12 minutes

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